


Triple Energy Window Scatter Correction - TEW

File Help

TEW Scatter Correction

About

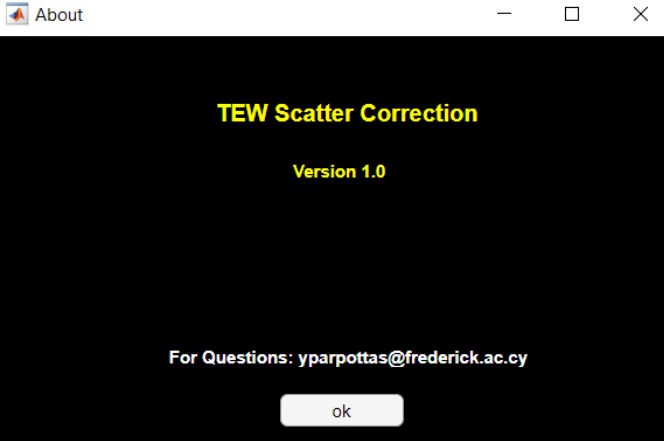

Main page

1

1. Enter the central energy (in keV) and central energy width (+/- %) for:


	Left Scatter Window	Main Photopeak	Right Scatter Window
Central Energy	<input type="text" value="306.9"/>	<input type="text" value="364"/>	<input type="text" value="421.1"/>
Central Energy Width	<input type="text" value="0.81"/>	<input type="text" value="10"/>	<input type="text" value="0.59"/>

2

2. **Calculate** energy widths

3

3. **Show** the energy widths

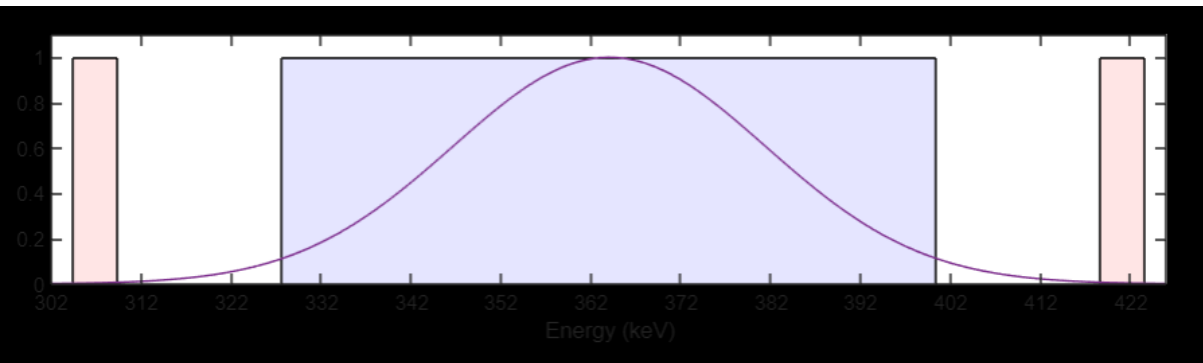

4

4. **Load** raw data for TEW correction

5

5. **Exit** output data in same folder ■